


PUREGRAFT Boost™ ADIPOSE MICRONIZER

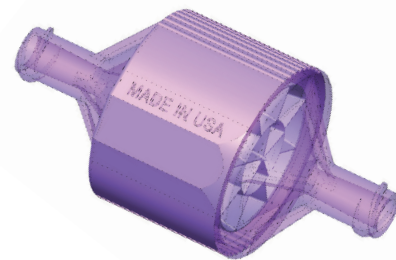

INSTRUCTIONS FOR USE

INTENDED USE

Puregraft Boost™ Adipose Micronizer is a proprietary patented single-use device designed to uniformly size harvested adipose tissue.


The Puregraft Boost can produce multiple-sized tissue derivatives from lipoaspirate:

1,500µm -500µm


NOTE: EACH PUREGRAFT BOOST CAN BE USED INDEPENDENTLY OR IN CONJUNCTION WITH OTHER SIZES, BUT DOES NOT REQUIRE A STEP DOWN / SIZE DOWN TECHNIQUE.

SIZING: 15TM PUREGRAFT BOOST ADIPOSE MICRONIZER (1,500µm)

1. Attach a sterile 20cc luer-lock syringe holding the adipose tissue to the 15TM Puregraft Boost (1,500µm).
2. Attach a second sterile 20cc luer-lock syringe to the other end of the 15TM Puregraft Boost.

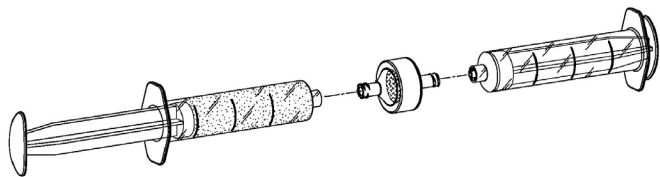

3. Manually force the adipose tissue back and forth between syringes 10 times to initiate sizing.

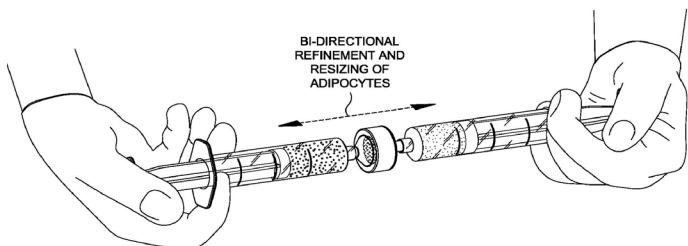

SIZING: 5TM PUREGRAFT BOOST ADIPOSE MICRONIZER (500µm)

1. Attach a sterile 20cc luer-lock syringe holding the adipose tissue to the 5TM Puregraft Boost (500µm).
2. Attach a second sterile 20cc luer-lock syringe to the other end of the 5TM Puregraft Boost.
3. Manually force the adipose tissue back and forth between syringes 10 times to initiate sizing.

QUICK START PROCESSING STEPS

ACQUIRE ADIPOSE GRAFT

- **Purify:** Utilizing a filtration method, wash lipoaspirate to remove infranatant fluid and contaminants from the adipose graft.
- **Fragment:** Attach syringe to each end of the Puregraft Boost and transfer specimen back and forth for 10 times.
- **Flush the Puregraft Boost:** Upon completion of transferring back and forth 10 times, use a 10ml syringe with 5ml of normal saline, attach an empty 10ml syringe and push the 5ml of saline through.

	Consult Instructions For Use	LOT Batch Code	STERILE R Sterilized Using Irradiation
	Do Not Reuse		Do Not Use If Damaged
REF	Catalog Number		Expiration Date
	Keep Dry		Manufacture Date
	Sterile Barrier		Do No Resterilize
Rx ONLY CAUTION: Federal law (USA) restricts this device to, by or on the order of a physician			

KMI IMI GROUP
(dba Innovative Med, Inc.)
4 Autry, Suite B
Irvine, CA 92618
949.458.1897



24-0082E